

October 7, 2022



## URGENT PRODUCT CORRECTION

---

<b>Description:</b>	<b>ENDURACHarge 36V Li-Ion Battery</b>
<b>Affected Serial Numbers:</b>	<b>See list within this notice</b>
<b>Date Code Range of Affected Product:</b>	<b>All ENDURACHarge 36V Li-Ion Batteries with a serial number bearing a batch code equal to or less than Year 21, Week 15. (Example: <u>20300102115M594</u>)</b>

---

Ferno has initiated a voluntary corrective action for specifically identified ENDURACHarge 36V Li-Ion batteries with a manufacturing date less than or equal to Week 15 of 2021. These batteries include units shipped with Ferno POWERX1 ambulance cots or purchased separately.

### Product Issue

Ferno has identified an issue wherein the ENDURACHarge 36V Li-Ion batteries could become unstable when not properly maintained and/or subjected to repeated drops, power washing directly into battery contacts or abuse. As stated in Ferno's instructions for use, a damaged battery can cause fire and/or injury.

### Ferno's Planned Action

Ferno will replace affected batteries with new batteries.

### Immediate Action

1. Immediately check the serial number of your Ferno POWER X1 Ambulance Cot (see list within this notice) as well as the serial number on the bottom of any ENDURACHarge 36V Li-Ion battery utilized with the cot. See 'How to Identify ENDURACHarge Battery Date Code'.
2. Check the serial number on the bottom of your ENDURACHarge 36V Li-Ion battery to see if the battery is in the identified date code range. If it is not, no further action is required and the battery may remain in service provided it does not have a compromised seal, damaged case, cracks, deformities, other visual damage or is hot to the touch. Regardless of date code, **if you identify a battery that has a compromised seal, damaged case, cracks, deformities, other visual damage or is hot to the touch, the battery should be immediately removed from service and moved outside your building.**
3. **ALL ENDURACHARGE 36V LI-ION BATTERIES MANUFACTURED WITHIN THE ABOVE REFERENCED DATE CODE RANGE MUST BE REPLACED. If you have a battery that is in the identified date code range, immediately visually inspect the battery. If you identify a battery that has a compromised seal, damaged case, cracks, deformities, other visual damage or is hot to the touch, the battery should be**

October 7, 2022

**immediately removed from service and moved outside your building. As stated in the battery manual, "A damaged battery can cause fire and/or injury. Inspect the battery before each use. If damaged, remove from service, recycle per the regulations in your country, and replace."**

**If you have a battery that is in the identified date code range and it does not display any wear or damage and is operating as intended, the following actions should be immediately taken if use is continued while awaiting a replacement battery:**

- Do not leave the battery installed in the cot when the cot is not in use**
- Do not leave the battery stored in the ambulance overnight**
- Only charge the battery during hours when it can be monitored**
- Only charge the battery in designated areas where no other combustibile items are located**

- Upon receipt of this Notice, please call Ferno Technical Support at 866-987-3776 or email [tscordinator@ferno.com](mailto:tscordinator@ferno.com) to (1) acknowledge receipt of this Urgent Product Correction notice, (2) advise of any serial numbers from the affected date code range in your inventory, (3) advise how many serial numbers are quarantined, (4) make arrangements to receive a replacement battery, and (5) discuss disposal/recycling of your affected battery.
- In the event you sold or donated the identified product, please forward this notice to your consignee and advise Ferno of the consignee information so we may follow up.

### **Other Information**



Ferno has submitted two Medical Device Reports with the FDA reporting thermal events wherein the batteries ignited. No reports of injury have been received.

Malfunctions or quality problems experienced with the use of ENDURACHarge 36V Li-Ion battery may be reported to Ferno at 866-987-3776 or [tscordinator@ferno.com](mailto:tscordinator@ferno.com) or FDA's MedWatch Adverse Event Reporting.

### **How to Identify ENDURACHarge Battery Date Code**

**The affected date code range is Week 15, Year 21 and all dates prior. See below examples:**

#### **1801 thru 2115**

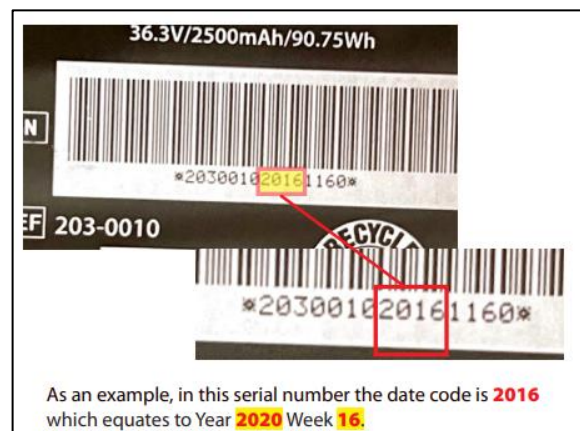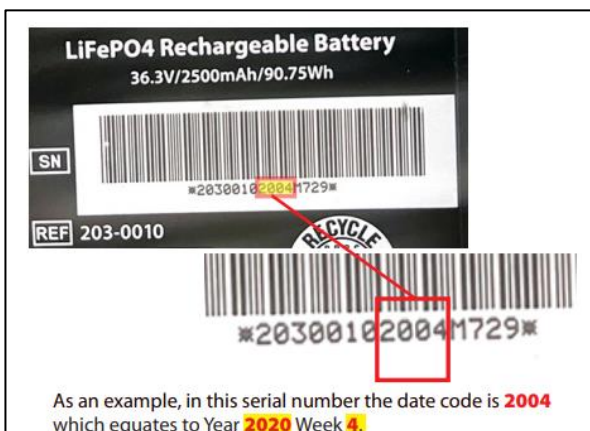

**Ferno POWER X1 Ambulance Cot Serial Numbers**

18F003862	18F003936	18F003981	19F004516	19F004564	19F004597	19F004646
18F003863	18F003937	18F003980	19F004517	19F004569	19F004622	19F004647
18F003864	18F003934	18F003982	19F004518	19F004565	19F004616	19F004649
18F003865	18F003935	18F003983	19F004522	19F004563	19F004620	19F004655
18F003866	18F003930	18F003979	19F004523	19F004566	19F004621	19F004656
18F003867	18F003925	18F003986	19F004524	19F004574	19F004624	19F004665
18F003868	18F003941	18F003984	19F004527	19F004572	19F004625	19F004663
18F003869	18F003940	18F003985	19F004525	19F004573	19F004619	19F004664
18F003870	18F003942	18F003987	19F004526	19F004575	19F004633	19F004669
18F003871	18F003944	18F003989	19F004528	19F004576	19F004634	19F004666
18F003872	18F003947	18F003999	19F004504	19F004571	19F004637	19F004668
18F003873	18F003946	18F003988	19F004537	19F004578	19F004635	19F004667
18F003874	18F003945	18F003990	19F004538	19F004577	19F004636	19F004674
18F003875	18F003943	18F003991	19F004505	19F004579	19F004623	19F004670
18F003876	18F003951	18F003998	19F004507	19F004580	19F004626	19F004671
18F003877	18F003950	18F004000	19F004508	19F004581	19F004627	19F004672
18F003878	18F003956	18F004001	19F004539	19F004582	19F004628	19F004673
18F003879	18F003957	18F004002	19F004506	19F004583	19F004629	19F004677
18F003880	18F003955	18F003993	19F004509	19F004584	19F004630	19F004681
18F003881	18F003952	18F003994	19F004548	19F004585	19F004631	19F004675
18F003882	18F003953	18F003992	19F004549	19F004586	19F004632	19F004676
18F003883	18F003958	18F003995	19F004540	19F004587	19F004654	19F004678
18F003884	18F003954	18F003997	19F004541	19F004588	19F004640	19F004679
18F003894	18F003959	18F003996	19F004552	19F004589	19F004641	19F004680
18F003893	18F003960	19F004501	19F004547	19F004590	19F004639	19F004682
18F003895	18F003961	19F004502	19F004553	19F004598	19F004638	19F004684
18F003896	18F003963	19F004503	19F004542	19F004599	19F004653	19F004685
18F003897	18F003962	18F004004	19F004554	19F004600	19F004650	19F004686
18F003898	18F003964	18F004007	19F004551	19F004601	19F004642	19F004683
18F003899	18F003965	18F004005	19F004550	19F004602	19F004643	19F004691
18F003900	18F003966	18F004006	19F004555	19F004603	19F004652	19F004687
18F003921	18F003967	18F004008	19F004556	19F004591	19F004648	19F004688
18F003922	18F003968	18F004009	19F004557	19F004592	19F004657	19F004689
18F003923	18F003969	19F004512	19F004558	19F004593	19F004658	19F004690
18F003924	18F003970	18F004010	19F004561	19F004594	19F004659	19F004707
18F003926	18F003974	19F004513	19F004562	19F004595	19F004660	19F004702
18F003927	18F003971	19F004515	19F004559	19F004596	19F004661	19F004703
18F003928	18F003973	19F004519	19F004560	19F004604	19F004651	19F004704
18F003931	18F003975	19F004520	19F004567	19F004615	19F004662	19F004706
18F003932	18F003976	19F004521	19F004568	19F004617	19F004644	19F004705
18F003933	18F003972	19F004514	19F004570	19F004618	19F004645	19F004713

October 7, 2022

19F004708	19F004772	19F004828	19F004895	19F004969	19F005001	19F005065
19F004709	19F004773	19F004829	19F004896	19F004941	19F005002	19F005068
19F004710	19F004774	19F004831	19F004835	19F004942	19F005003	19F005069
19F004715	19F004775	19F004832	19F004818	19F004943	19F005006	19F005070
19F004711	19F004776	19F004830	19F004819	19F004944	19F005007	19F005071
19F004712	19F004777	19F004836	19F004820	19F004955	19F005004	19F005072
19F004714	19F004778	19F004837	19F004821	19F004934	19F005012	19F005067
19F004716	19F004779	19F004838	19F004897	19F004945	19F005008	19F005073
19F004733	19F004780	19F004839	19F004817	19F004957	19F005009	19F005074
19F004734	19F004781	19F004840	19F004898	19F004959	19F005010	19F005076
19F004717	19F004782	19F004841	19F004902	19F004956	19F005011	19F005075
19F004718	19F004783	19F004842	19F004872	19F004958	19F005015	19F005079
19F004732	19F004786	19F004843	19F004901	19F004961	19F005016	19F005080
19F004719	19F004784	19F004844	19F004899	19F004962	19F005020	19F005081
19F004720	19F004785	19F004845	19F004903	19F004963	19F005027	19F005077
19F004721	19F004794	19F004846	19F004900	19F004964	19F005018	19F005083
19F004722	19F004803	19F004847	19F004905	19F004965	19F005021	19F005084
19F004723	19F004804	19F004848	19F004907	19F004966	19F005022	19F005078
19F004724	19F004802	19F004849	19F004904	19F004967	19F005029	19F005082
19F004725	19F004801	19F004850	19F004908	19F004968	19F005013	19F005091
19F004726	19F004796	19F004851	19F004906	19F004971	19F005014	19F005092
19F004727	19F004795	19F004852	19F004909	19F004972	19F005019	19F005093
19F004728	19F004797	19F004853	19F004910	19F004973	19F005017	19F005094
19F004729	19F004798	19F004854	19F004931	19F004974	19F005030	19F005086
19F004730	19F004799	19F004855	19F004932	19F004975	19F005023	19F005087
19F004731	19F004805	19F004856	19F004933	19F004976	19F005024	19F005095
19F004737	19F004806	19F004857	19F004823	19F004977	19F005025	19F005088
19F004738	19F004807	19F004858	19F004936	19F004978	19F005026	19F005089
19F004739	19F004808	19F004859	19F004893	19F004981	19F005031	19F005090
19F004740	19F004809	19F004860	19F004935	19F004982	19F005032	19F005096
19F004741	19F004810	19F004861	19F004938	19F004960	19F005033	19F005097
19F004744	19F004811	19F004862	19F004939	19F004980	19F005028	19F005098
19F004736	19F004812	19F004863	19F004937	19F004993	19F005051	19F005099
19F004742	19F004813	19F004864	19F004953	19F004970	19F005048	19F005100
19F004735	19F004800	19F004865	19F004954	19F004979	19F005050	19F005101
19F004746	19F004834	19F004866	19F004948	19F004998	19F005045	19F005102
19F004743	19F004814	19F004867	19F004949	19F004994	19F005046	19F005103
19F004745	19F004815	19F004868	19F004950	19F004995	19F005047	19F005104
19F004767	19F004816	19F004869	19F004951	19F004997	19F005049	19F005105
19F004770	19F004824	19F004870	19F004952	19F004999	19F005064	19F005106
19F004768	19F004825	19F004871	19F004940	19F005005	19F005062	19F005107
19F004771	19F004826	19F004822	19F004946	19F004996	19F005066	19F005108
19F004769	19F004827	19F004894	19F004947	19F005000	19F005063	19F005109

October 7, 2022

19F005110	19F005163	19F005191	19F005256	20F006372	20F006425	20F006482
19F005111	19F005164	19F005194	19F005257	20F006373	20F006429	20F006485
19F005112	19F005165	19F005192	19F005242	20F006374	20F006430	20F006486
19F005113	19F005166	19F005190	19F005258	20F006375	20F006431	20F006484
19F005114	19F005167	19F005221	19F005259	20F006312	20F006433	20F006483
19F005115	19F005168	19F005219	19F005251	20F006313	20F006435	20F006487
19F005034	19F005169	19F005220	19F005252	20F006376	20F006436	20S019635
19F005035	19F005170	19F005222	20F006300	20F006377	20F006438	20S019631
19F005036	19F005153	19F005223	20F006301	20F006378	20F006432	20S019632
19F005116	19F005159	19F005224	19F005253	20F006379	20F006437	20S019634
19F005117	19F005160	19F005225	20F006308	20F006380	20F006446	20S019637
19F005118	19F005174	19F005226	20F006302	20F006381	20F006434	20S019638
19F005119	19F005175	19F005227	20F006309	20F006382	20F006447	20S019633
19F005120	19F005173	19F005217	20F006310	20F006383	20F006441	20S019688
19F005121	19F005172	19F005215	20F006311	20F006384	20F006442	20S019682
19F005122	19F005176	19F005214	20F006303	20F006385	20F006443	20S019686
19F005123	19F005177	19F005228	20F006304	20F006388	20F006448	20S019687
19F005124	19F005171	19F005229	20F006305	20F006386	20F006445	20S019683
19F005125	19F005178	19F005213	20F006306	20F006391	20F006450	20S019684
19F005136	19F005179	19F005216	20F006307	20F006392	20F006451	20S019685
19F005137	19F005182	19F005218	20F006339	20F006387	20F006452	20S019762
19F005138	19F005183	19F005211	20F006345	20F006390	20F006453	20S019764
19F005139	19F005184	19F005212	20F006344	20F006389	20F006444	20S019639
19F005145	19F005185	19F005230	20F006340	20F006393	20F006461	20S019763
19F005144	19F005187	19F005231	20F006341	20F006394	20F006449	20S019771
19F005142	19F005188	19F005232	20F006342	20F006395	20F006454	20S019640
19F005140	19F005180	19F005233	20F006343	20F006397	20F006462	20S019767
19F005143	19F005181	19F005234	20F006346	20F006402	20F006463	20S019766
19F005152	19F005204	19F005235	20F006362	20F006396	20F006464	20S019765
19F005148	19F005186	19F005210	20F006363	20F006401	20F006465	20S019773
19F005149	19F005205	19F005243	20F006364	20F006403	20F006466	20S019816
19F005151	19F005206	19F005236	20F006359	20F006398	20F006467	20S019770
19F005158	19F005201	19F005260	20F006347	20F006399	20F006468	20S019818
19F005141	19F005199	19F005244	20F006358	20F006404	20F006469	20S019817
19F005157	19F005200	19F005245	20F006365	20F006400	20F006455	20S019772
19F005146	19F005203	19F005246	20F006366	20F006405	20F006471	20S019819
19F005147	19F005202	19F005247	20F006360	20F006406	20F006456	20S019768
19F005150	19F005189	19F005248	20F006361	20F006424	20F006470	20S019820
19F005154	19F005195	19F005249	20F006368	20F006426	20F006472	20F006488
19F005155	19F005198	19F005250	20F006367	20F006428	20F006473	20F006489
19F005156	19F005196	19F005261	20F006369	20F006422	20F006474	20F006490
19F005161	19F005197	19F005254	20F006370	20F006423	20F006480	20S019769
19F005162	19F005193	19F005255	20F006371	20F006427	20F006481	20S019845

October 7, 2022

20S019848	20S019901	20S019978	20S020037	20S020110	20S020166	20S020315
20S019853	20S019899	20S019984	20S020039	20S020111	20S020171	20S020316
20S019850	20S019902	20S019974	20S020044	20S020112	20S020167	20S020317
20S019851	20S019911	20S019975	20S020045	20S020135	20S020170	20S020203
20S019852	20S019912	20S019977	20S020046	20S020098	20S020202	20S020231
20S019844	20S019913	20S019966	20S020040	20S020131	20S020172	20S020232
20S019847	20S019904	20S019985	20S020041	20S020132	20S020173	20S020233
20S019854	20S019903	20S019986	20S020042	20S020133	20S020174	20S020300
20S019855	20S019914	20S019987	20S020043	20S020134	20S020175	20S020301
20S019849	20S019919	20S019970	20S020076	20S020155	20S020176	20S020271
20S019846	20S019918	20S019994	20S020077	20S020148	20S020169	20S020305
20S019878	20S019915	20S019995	20S020075	20S020154	20S020168	20S020306
20S019879	20S019916	20S019997	20S020078	20S020113	20S020214	20S020298
20S019880	20S019917	20S019996	20S020079	20S020114	20S020216	20S020199
20S019881	20S019905	20S019998	20S020080	20S020115	20S020218	20S020267
20S019882	20S019906	20S019999	20S020081	20S020116	20S020204	20S020268
20S019883	20S019933	20S020000	20S020083	20S020149	20S020213	20S020269
20S019884	20S019936	20S020001	20S020082	20S020150	20S020215	20S020270
20S019885	20S019931	20S020002	20S020084	20S020151	20S020217	20S020299
20S019887	20S019907	20S020003	20S020085	20S020177	20S020219	20S020272
20S019886	20S019910	20S020004	20S020086	20S020117	20S020220	20S020273
20S019889	20S019934	20S020005	20S020087	20S020118	20S020221	20S020274
20S019890	20S019908	20S020017	20S020088	20S020119	20S020223	20S020275
20S019892	20S019909	20S020012	20S020089	20S020120	20S020225	20S020179
20S019868	20S019937	20S020018	20S020090	20S020121	20S020276	20S020280
20S019869	20S019929	20S020013	20S020091	20S020122	20S020278	20S020281
20S019870	20S019932	20S020014	20S020092	20S020123	20S020222	20S020302
20S019871	20S019935	20S020015	20S020093	20S020124	20S020226	20S020297
20S019872	20S019930	20S020019	20S020094	20S020125	20S020277	20S020318
20S019873	20S019958	20S020006	20S020095	20S020126	20S020279	20S020304
20S019874	20S019959	20S020007	20S020096	20S020152	20S020224	20S020303
20S019875	20S019960	20S020008	20S020097	20S020153	20S020227	20S020344
20S019876	20S019961	20S020009	20S020099	20S020163	20S020228	20S020319
20S019888	20S019964	20S020016	20S020100	20S020156	20S020229	20S020336
20S019877	20S019963	20S020010	20S020101	20S020157	20S020230	20S020342
20S019891	20S019973	20S020011	20S020102	20S020164	20S020307	20S020360
20S019893	20S019962	20S020032	20S020103	20S020165	20S020308	20S020348
20S019894	20S019965	20S020031	20S020104	20S020178	20S020309	20S020341
20S019895	20S019976	20S020033	20S020105	20S020158	20S020310	20S020355
20S019896	20S019971	20S020038	20S020106	20S020159	20S020311	20S020345
20S019898	20S019972	20S020034	20S020107	20S020162	20S020312	20S020346
20S019897	20S019982	20S020035	20S020108	20S020160	20S020313	20S020361
20S019900	20S019983	20S020036	20S020109	20S020161	20S020314	20S020356

October 7, 2022

20S020357	20S020470	20S020530	20S020338	21S023642	21S023797	
20S020354	20S020471	20S020531	20S020343	21S023643	21S023798	21S023849
20S020347	20S020472	20S020532	20S020339	21S023644	21S023799	21S023851
20S020349	20S020473	20S020533	20S020200	21S023594	21S023709	21S023854
20S020350	20S020474	20S020534	20S020201	21S023595	21S023710	21S023918
20S020351	20S020475	21S023344	21S023349	21S023585	21S023703	21S023907
20S020352	20S020476	21S023338	21S023350	21S023586	21S023708	21S023912
20S020353	20S020477	21S023343	21S023466	21S023587	21S023706	21S023906
20S020417	20S020478	20S020535	21S023465	21S023588	21S023707	21S023908
20S020358	20S020479	20S020536	21S023391	21S023589	21S023811	21S023909
20S020359	20S020480	20S020537	21S023461	21S023590	21S023812	21S023910
20S020424	20S020481	20S020538	21S023462	21S023591	21S023813	21S023911
20S020418	20S020482	20S020539	21S023463	21S023592	21S023814	21S023913
20S020416	20S020483	20S020540	21S023464	21S023593	21S023815	21S023914
20S020425	20S020484	20S020541	21S023460	21S023597	21S023816	21S023915
20S020426	20S020485	21S023339	21S023348	21S023645	21S023817	21S023919
20S020422	20S020486	21S023340	21S023469	21S023656	21S023818	21S023916
20S020421	20S020487	21S023341	21S023467	21S023649	21S023856	21S023950
20S020423	20S020488	21S023342	21S023468	21S023651	21S023808	21S023951
20S020461	20S020490	21S023351	21S023470	21S023650	21S023809	21S023917
20S020463	20S020491	20S020542	21S023471	21S023646	21S023810	21S023940
20S020462	20S020492	20S020543	21S023474	21S023647	21S023819	21S023941
20S020419	20S020493	20S020544	21S023475	21S023648	21S023820	21S023942
20S020420	20S020466	21S023345	21S023476	21S023655	21S023822	21S023988
20S020457	20S020340	21S023346	21S023477	21S023652	21S023821	21S023989
20S020456	20S020551	21S023386	21S023473	21S023653	21S023800	21S023990
20S020450	20S020452	21S023387	21S023472	21S023654	21S023801	21S023991
20S020464	20S020453	21S023388	21S023478	21S023702	21S023802	21S023943
20S020458	20S020454	21S023389	21S023512	21S023699	21S023803	21S023944
20S020451	20S020455	21S023376	21S023514	21S023698	21S023804	21S023945
20S020465	20S020546	21S023377	21S023520	21S023700	21S023807	21S023946
20S020459	20S020547	21S023390	21S023596	21S023701	21S023805	21S023947
20S020460	20S020549	21S023378	21S023516	21S023704	21S023838	21S023948
20S020512	20S020550	21S023379	21S023517	21S023705	21S023839	21S023949
20S020513	20S020548	21S023380	21S023519	21S023711	21S023840	21S023992
20S020514	20S020545	21S023381	21S023515	21S023712	21S023841	21S023987
20S020515	21S023336	21S023382	21S023518	21S023791	21S023806	21S023996
20S020516	20S020527	21S023383	21S023511	21S023792	21S023853	21S023993
20S020489	21S023335	21S023384	21S023513	21S023793	21S023842	21S023994
20S020467	20S020528	21S023385	21S023639	21S023794	21S023843	21S023995
20S020468	20S020529	21S023347	21S023640	21S023795	21S023852	
20S020469	21S023337	20S020337	21S023641	21S023796	21S023850	

October 7, 2022

21S023996	21S024069	21S024225	21S024316	21S024455	21S024529	21S024584
21S023997	21S024070	21S024226	21S024317	21S024456	21S024530	21S024585
21S023998	21S024071	21S024227	21S024318	21S024457	21S024531	21S024586
21S023999	21S024072	21S024228	21S024319	21S024458	21S024532	21S024587
21S024000	21S024137	21S024229	21S024320	21S024474	21S024533	21S024588
21S024001	21S024138	21S024230	21S024321	21S024475	21S024534	21S024589
21S024002	21S024139	21S024231	21S024322	21S024476	21S024535	21S024590
21S024027	21S024140	21S024232	21S024350	21S024477	21S024536	21S024591
21S024028	21S024141	21S024233	21S024351	21S024490	21S024537	21S024592
21S024029	21S024142	21S024234	21S024352	21S024491	21S024538	21S024593
21S024030	21S024155	21S024235	21S024353	21S024492	21S024539	21S024594
21S024031	21S024156	21S024236	21S024378	21S024493	21S024540	21S024595
21S024032	21S024157	21S024237	21S024379	21S024494	21S024541	21S024596
21S024033	21S024158	21S024238	21S024380	21S024495	21S024542	21S024597
21S024034	21S024159	21S024239	21S024381	21S024496	21S024543	21S024598
21S024035	21S024160	21S024240	21S024382	21S024497	21S024544	21S024599
21S024036	21S024161	21S024241	21S024383	21S024498	21S024545	21S024600
21S024037	21S024162	21S024242	21S024384	21S024499	21S024546	21S024601
21S024038	21S024163	21S024243	21S024385	21S024500	21S024547	21S024602
21S024039	21S024164	21S024244	21S024386	21S024501	21S024548	21S024603
21S024040	21S024165	21S024245	21S024387	21S024502	21S024555	21S024604
21S024041	21S024166	21S024246	21S024388	21S024507	21S024556	21S024605
21S024042	21S024167	21S024247	21S024389	21S024508	21S024557	21S024606
21S024043	21S024168	21S024248	21S024390	21S024509	21S024558	21S024632
21S024044	21S024191	21S024249	21S024391	21S024510	21S024559	21S024633
21S024045	21S024192	21S024250	21S024417	21S024511	21S024560	21S024634
21S024046	21S024193	21S024251	21S024438	21S024512	21S024567	21S024635
21S024053	21S024194	21S024252	21S024439	21S024513	21S024568	21S024636
21S024054	21S024195	21S024253	21S024440	21S024514	21S024569	21S024637
21S024055	21S024199	21S024254	21S024441	21S024515	21S024570	21S024638
21S024056	21S024210	21S024255	21S024442	21S024516	21S024571	21S024639
21S024057	21S024211	21S024256	21S024443	21S024517	21S024572	21S024640
21S024058	21S024212	21S024257	21S024444	21S024518	21S024573	21S024641
21S024059	21S024213	21S024258	21S024445	21S024519	21S024574	21S024642
21S024060	21S024214	21S024259	21S024446	21S024520	21S024575	21S024643
21S024061	21S024215	21S024260	21S024447	21S024521	21S024576	21S024644
21S024062	21S024216	21S024309	21S024448	21S024522	21S024577	21S024645
21S024063	21S024217	21S024310	21S024449	21S024523	21S024578	21S024646
21S024064	21S024218	21S024311	21S024450	21S024524	21S024579	21S024647
21S024065	21S024219	21S024312	21S024451	21S024525	21S024580	21S024648
21S024066	21S024220	21S024313	21S024452	21S024526	21S024581	21S024649
21S024067	21S024223	21S024314	21S024453	21S024527	21S024582	21S024650
21S024068	21S024224	21S024315	21S024454	21S024528	21S024583	21S024651



October 7, 2022

21S024652	21S024672	21S024735	21S024756	21S024776	21S024805	21S024825
21S024653	21S024673	21S024736	21S024757	21S024779	21S024806	21S024842
21S024654	21S024674	21S024737	21S024758	21S024780	21S024807	21S024843
21S024655	21S024710	21S024738	21S024759	21S024781	21S024808	21S024844
21S024656	21S024711	21S024739	21S024760	21S024782	21S024809	21S024845
21S024657	21S024712	21S024740	21S024761	21S024783	21S024810	21S024846
21S024658	21S024713	21S024741	21S024762	21S024784	21S024811	21S024847
21S024659	21S024714	21S024742	21S024763	21S024785	21S024812	21S024848
21S024660	21S024715	21S024743	21S024764	21S024786	21S024813	21S024856
21S024661	21S024716	21S024744	21S024765	21S024787	21S024814	21S024857
21S024662	21S024717	21S024745	21S024766	21S024789	21S024815	21S024858
21S024663	21S024718	21S024746	21S024767	21S024790	21S024816	21S024859
21S024664	21S024719	21S024747	21S024768	21S024791	21S024817	21S024866
21S024665	21S024720	21S024748	21S024769	21S024798	21S024818	21S024867
21S024666	21S024721	21S024749	21S024770	21S024799	21S024819	21S024868
21S024667	21S024730	21S024750	21S024771	21S024800	21S024820	
21S024668	21S024731	21S024751	21S024772	21S024801	21S024821	
21S024669	21S024732	21S024752	21S024773	21S024802	21S024822	
21S024670	21S024733	21S024754	21S024774	21S024803	21S024823	
21S024671	21S024734	21S024755	21S024775	21S024804	21S024824	