

FERNO

S-CAPEPOD® & S-CAPEPLUS®

Evacuation mattresses and sheets

For the quick and safe evacuation of bed-bound and immobile people in an emergency – **where speed and ease of deployment is critical**



S-CAPEPLUS
Dedicated vertical
evacuation mattress



S-CAPEPLUS Premium
with integrated head restraint
and foot support



S-CAPEPOD
Bed-bound evacuation: Evac
sheet stored under the bed

FOR MORE DETAILED
INFORMATION SEE OVER

Specifications

FERNO – is a world leading innovator and manufacturer of patient transport solutions

Our product range is born of a closer understanding of our customers' needs and the practical demands of their work.

Ferno's industry-leading **S-CAPEPOD evacuation sheets** and **S-CAPEPLUS evacuation mattresses** typify our passion for top quality, highly functional design patient transfer products that don't just do the job, they do it **BETTER**.

S-CAPEPOD Evac Sheet	Folded	Under mattress	Flaps Open	Weight	Load Limit	To Fit Mattress Size
S-CAPEPOD Budget	400x300 mm	2050x780 mm	n/a	1.0 kg	150 kg (120kg*)	Up to 2100 mm, 800-900 mm width
S-CAPEPOD Standard	400x300 mm	2100x840 mm	3100x2600 mm	1.1 kg	150 kg (120kg*)	Up to 2100 mm, 800-900 mm width
*S-CAPEPOD Budget and Standard tested to 150 kgs but where patient has large/extended physical mid/lower-body mass, max SWL 120 kg as greater may not physically fit						
S-CAPEPOD Bariatric	480x300 mm	2100x1020 mm	3570x3090 mm	2.5 kg	300 kg	Up to 2160 mm, 1000-1250 mm width

S-CAPEPLUS Evac Mattress	In carry bag	Mattress size	Dims Flaps Open	Weight	Load Limit
S-CAPEPLUS Standard	600x350x600 mm	2100*600 mm	1900x2850 mm	5.5 kg	200 kg
S-CAPEPLUS Youth	600x300x600 mm	1750*600 mm	1900x2300 mm	3.5 kg	200 kg
S-CAPEPLUS Premium	600x350x600 mm	2100*600 mm	1900x2850 mm	6.5 kg	200 kg
S-CAPEPLUS Bariatric	600*430*900 mm	2030*850 mm		14 kg	400 kg
S-CAPEKIDS for babies	950x130x660mm	1960*660 mm	n/a	6.5 kg	4 pockets, 15 kg / pocket. Approx max height 90cm

call +44 (0) 1274 851 999
email sales.uk@ferno.com
www ferno.com



FERNO

S-CAPEPOD® & S-CAPEPLUS®

For the quick and safe evacuation of bed-bound and immobile people in an emergency – where speed and ease of deployment is critical.

S-CAPEPOD is a simple-to-deploy evacuation device, stored below the bed mattress which, when required, can be simply unfolded and fitted around the person laid on the bed mattress.

The unique design of using folding flap sections at each side and at the foot-end of the sheet creates a sturdy 'bag' in which to envelop the patient and mattress. Using heavy-duty hook & loop fixation system to hold it all together, once deployed it gives an extremely fast, safe and effective system for the evacuation of a bed-bound patient.

Hand-hold straps at the head and foot-end allow the mattress and occupant to be easily manoeuvred along the floor and down stairs. Made of a polyester material, the S-CAPEPOD sheet glides easily during evacuation, as well as being breathable when stored under the mattress and is also easily washable. Made from anti-bacterial materials, infection control has also been considered.

Ideal for Hospitals and Care Homes when in an emergency situation where a high number of bed-bound people may need to be evacuated directly from their beds, S-CAPEPOD simply sits under the bed mattress and can be deployed in seconds.

S-CAPEPLUS is based around the same concept as S-CAPEPOD, but is a dedicated evacuation mattress with a 8cm thick base mattress incorporated within the device, allowing patients to be quickly and easily transferred downstairs laid down. Used in combination with Ferno's range of evacuation chairs, it ensures that people of all sizes and conditions can be effectively evacuated in an emergency, seated or recumbent.

- **Fast and safe** – you are in control of the evacuation of non-mobile people
- **Extremely easy to use** – two-person operation is advised but in extreme emergency can be operated by one person only. Requires very little training.
- **Various models for differing needs** – **S-CAPEKIDS** for evacuation of babies, **S-CAPEPLUS YOUTH** for younger children, **S-CAPEPLUS STANDARD** for children and adults, **S-CAPEPLUS PREMIUM** for the ultimate in safe evacuation, with built-in head restraint, footrest and increased durability materials.
- **Virtually no maintenance** – strong, hard-wearing and easy to clean, once installed an annual inspection is all that is required.

Part Numbers

50832100AZ 'S-CAPEPOD BUDGET' under-bed evacuation sheet, 120-150 kg SWL
 5N88888813 'S-CAPEPOD STANDARD' under-bed evacuation sheet, 150 kg SWL
 5N88888812 'S-CAPEPOD BARIATRIC' under-bed evacuation sheet, 300 kg SWL
 5N8888SCKW 'S-CAPEKIDS' evacuation mattress, evacuate up to 4 babies, 15kg SWL each pocket
 5N88888750 'S-CAPEPLUS YOUTH' evacuation mattress, SWL 200 kg
 5NN8888886 'S-CAPEPLUS STANDARD' evacuation mattress, SWL 200 kg
 5N88888866 'S-CAPEPLUS BARIATRIC' evacuation mattress, SWL 400 kg
 5NOUT88886 'S-CAPEPLUS PREMIUM' evacuation mattress incl. headblocks, SWL 200 kg
 5N88888102 Wall storage cover for S-CAPEPLUS, Green
 5735945281 Storage Cabinet for S-CAPEPLUS



S-CAPEPOD stored under bed mattress



Deploying S-CAPEPOD



Evacuating with S-CAPEPOD



S-CAPEKIDS



S-CAPEPLUS STANDARD



S-CAPEPLUS PREMIUM

FERNO - S-CAPE FPC015 09.24

call +44 (0) 1274 851 999
email sales.uk@ferno.com
www ferno.com



Ferno (UK) Limited, Stubbs Beck Lane, Cleckheaton, West Yorkshire, BD19 4TZ, England